



ChairPro™ / MultiPro™ Seat Belt Alarm

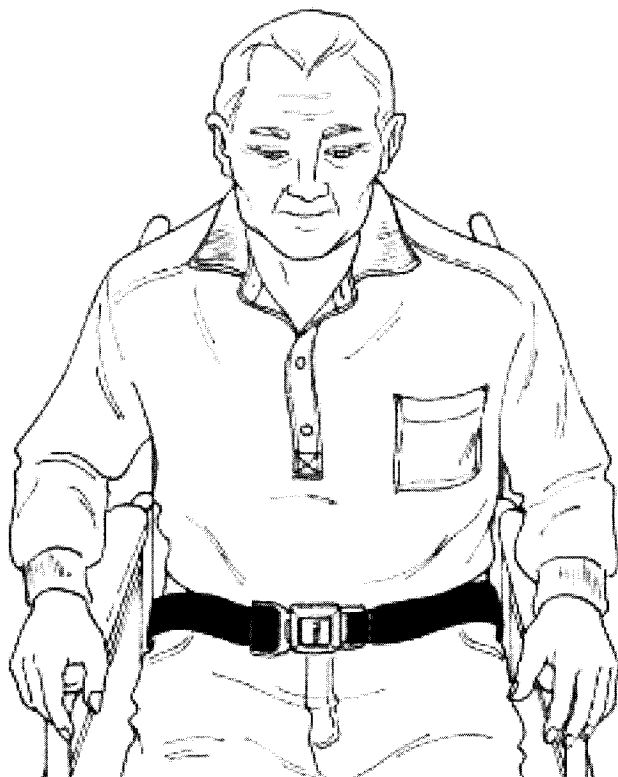


4. Insert mini-phone plug into Mini-Phone plug receptacle on right side of alarm. (Illustration 2) Alarm will sound. Press Auto-Reset to silence alarm.

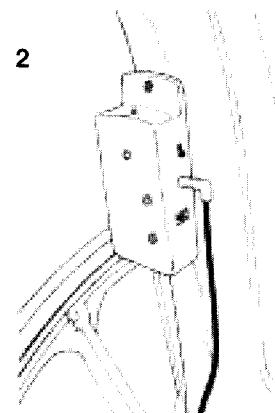

OPERATION


1. Use the following procedure to test the system before each use.
 - a. Press Battery-Test button. If alarm is weak or fails to sound, replace 9-volt battery and press button again. If alarm is still weak or fails to sound, discontinue use of alarm.
 - b. Seat patient and secure buckle. Battery indicator light should flash. If light fails to flash, replace battery and make certain that the spade connectors and mini-phone plug are properly connected. Retest. If battery indicator light still fails to flash, discontinue use of system.
 - c. Release buckle. Alarm will sound. Secure buckle. Alarm will silence itself. It is now in the monitoring mode. If alarm fails to sound when buckle is released, check connection points and/or replace battery. Retest. If alarm again fails to sound, discontinue use of system.

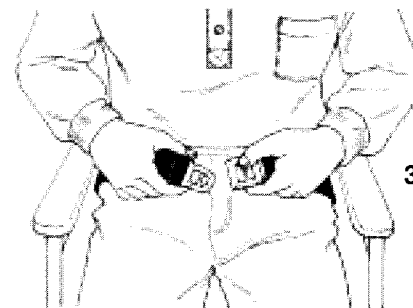

Model #	Description	Guarantee	Unit
909392	Seat Belt Alarm	One Year	Each

INSTALLATION

1. Attach the two "spade" connectors on the connector wire to those on the seat belt. (Illustration 1)
2. Remove rearmost screws from both sides of wheelchair seat. Remove grommets from screws. Place screws through grommets at the ends of both belt sections and refasten screws to the wheelchair seat. (Illustration 1)
3. Place connector cord through the space between the wheelchair seat and backrest. Make certain that it is away from wheels and other moving parts.

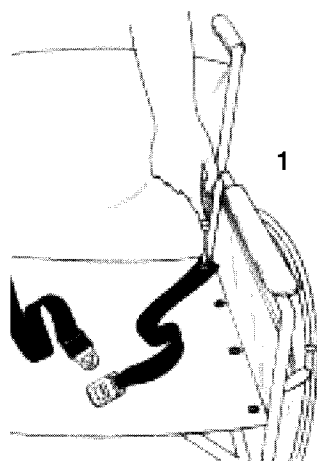

2. Select high or low volume.
3. Alarm is activated when patient releases the buckle. Press the Auto-Reset button to silence the alarm. The alarm is now in stand-by mode. Do not secure the buckle when the system is not being used to monitor a patient. Securing the buckle will cause the battery indicator light to flash and shorten battery life.
4. The alarm will automatically go into the monitoring mode when the buckle is secured. **There is no on/off switch.**
5. After the buckle is secured, adjust the belt to a proper fit around patient's hips. The belt is released by pushing the Press button. (Illustration 3)