
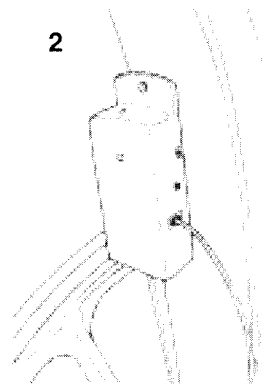


# ChairPro<sup>™</sup> / MultiPro<sup>™</sup> with E-Z Release Velcro<sup>®</sup> Belt Alarm

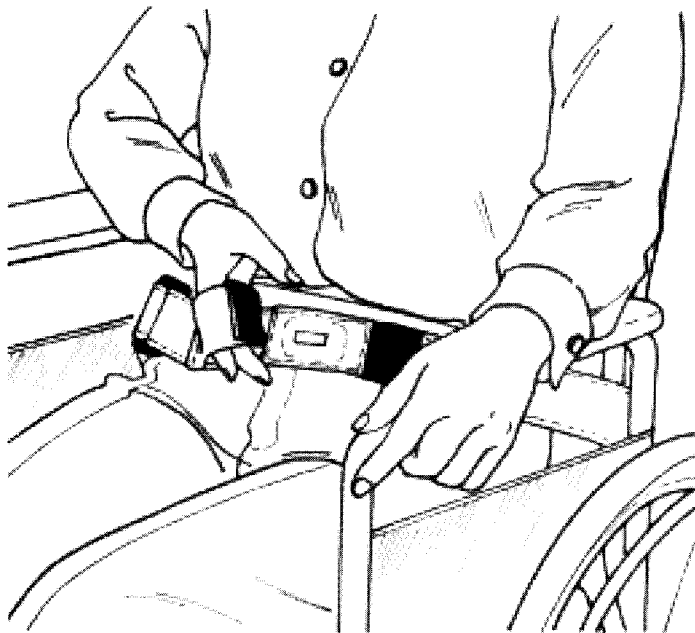


3. Insert telephone-style plug into Modular Receptacle on right side of alarm. (Illustration 2) Alarm will sound. Press Auto-Reset to silence alarm.

Model #	Description	Guarantee	Unit
909393	E-Z Release Velcro <sup>®</sup> Belt Alarm	One Year	Each

## INSTALLATION

1. Remove the rearmost screws from the left and right sides of the wheelchair seat. Insert the screws through the grommets on the ends of the two belt sections. Refasten screws to the wheelchair. (Illustration 1)

2. Place the telephone-style connector cord through the space between the wheelchair seat and backrest. Make certain that it is away from wheels and other moving parts.

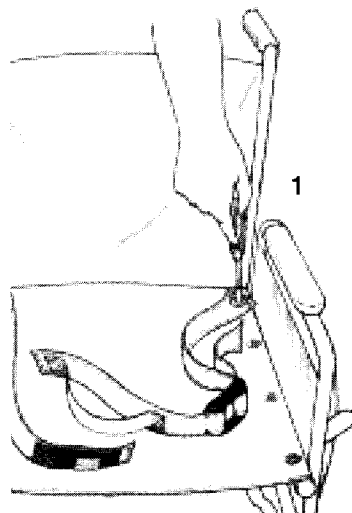

## OPERATION

1. Use the following procedure to test the system before each use.

a. Press Battery-Test button. If alarm is weak or fails to sound, replace 9-volt battery and press button again. If alarm is still weak or fails to sound, discontinue use of alarm.

b. Seat patient and secure belt by placing the magnet (located under red loop) over metal strip. (Illustration 3) Battery indicator light should flash. If light fails to flash, replace battery and make certain that the telephone-style plug is securely connected to the alarm unit. Retest. If battery indicator light still fails to flash, discontinue use of system.

c. Open magnetic closure. Alarm will sound. Secure closure. Alarm will silence itself. It is now in the monitoring mode. If alarm fails to sound when magnetic closure is opened, check plug connection and/or replace battery. Retest. If alarm again fails to sound, discontinue use of system.

2. Select high or low volume.

3. Alarm is activated when patient opens the magnetic closure. Press the Auto-Reset button to silence the alarm. The alarm is now in stand-by mode. Do not secure the magnetic closure when the system is not being used to monitor a patient. Securing the closure will cause the battery indicator light to flash and shorten battery life.

4. The alarm will automatically go into the monitoring mode when the magnetic closure is secured. **There is no on/off switch.**

5. After the closure is secured, adjust the belt to a proper fit around patient's hips. The wheelchair user may easily release the belt by placing his/her hand through the red loop and pulling up on the end.

