


## Foot-Drop Boot

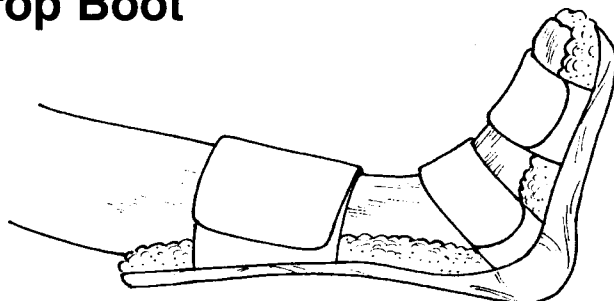

Reorder #	Description	Size	Shoe Size	Unit
<input type="checkbox"/> 503170	Foot-Drop Boot	Small	Men's 5-7, Women's 3-6	Each
<input type="checkbox"/> 503171	Foot-Drop Boot	Medium	Men's 7-9, Women's 6-9	Each
<input type="checkbox"/> 503172	Foot-Drop Boot	Large	Men's 9 -12, Women's 9-11	Each
<input type="checkbox"/> 503180	Replacement Cover	Small	Men's 5-7, Women's 3-6	Each
<input type="checkbox"/> 503181	Replacement Cover	Medium	Men's 7-9, Women's 6-9	Each
<input type="checkbox"/> 503182	Replacement Cover	Large	Men's 9 -12, Women's 9-11	Each

### Purpose

Designed with a supportive plastic inner splint that controls plantar flexion to reduce foot drop. Patient's heel is suspended above a molded cavity for maximum pressure reduction. Synthetic sheepskin liner adds comfort while providing protection against skin shear.

### Application

1. Select proper size for patient. See size chart above.
2. Place patient's foot in boot so that heel is suspended over cavity.
3. Secure Velcro<sup>®</sup> straps.

### Laundering

Remove cover. Do not launder inner splint. The Foot-Drop Boot cover may be laundered and dried using standard white goods procedures. Do not wash or dry at temperatures above 180°F (82° C). The use of bleach is not recommended.