


## GERI-CHAIR SMOKER'S APRON


| Reorder # | Color | Fabric              | Unit |
|-----------|-------|---------------------|------|
| 906015    | White | Skil-Care Firegard™ | Each |
| 906016    | Blue  | Skil-Care Firegard™ | Each |



**A**

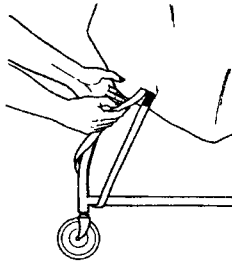

**B**

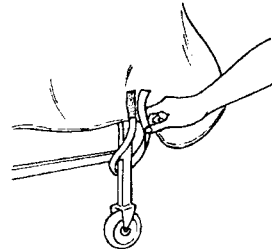

### Purpose

Skil-Care's Geri-Chair Smoker's Apron is designed as an aid in the prevention of accidental ignition of a resident's or patient's clothing. It is not a substitute for proper supervision. Patients or residents in geri-chairs who smoke must be supervised. The Geri-Chair Smoker's Apron is intended for geri-chair use only. If the smoker is in a wheelchair, we suggest the Smoker's Apron designed for wheelchair use. The model numbers are 906010 (white) and 906011 (blue pattern). See reverse side.

### Easy to Use

1. Place the apron over resident/patient and adjust the neck strap so that the top of the apron is above the chest. See illustration above.
2. Secure upper Velcro<sup>®</sup> closures on left and right sides above left and right rear wheels. See illustration A.
3. Secure lower Velcro<sup>®</sup> closures on left and right sides above left and right front wheels. See illustration B.

### Easy Care

The Smoker's Apron may be cleaned with a spray-type cleanser and a damp cloth or sponge. Do not launder. This apron is made of Skil-Care Firegard™ fabric, which can be gently folded for storage without the risk of developing cracks or pinholes.