


Head Positioner

Reorder # 703121



PURPOSE

Provides comfortable lateral head support. Designed to prevent hyperextension of the neck. Attaches to Skil-Care's Reclining Wheelchair Backrest and Air Flo Reclining Backrest as well as to all extended-back wheelchairs, geri-chairs, and recliners.

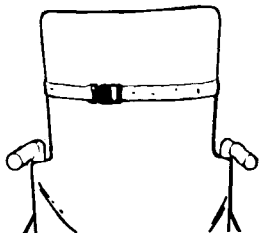

APPLICATION

1. Place Head Positioner on backrest so that it is just above the patient's shoulders. (See illustration above.)
2. Bring strap behind backrest and secure buckle. Tighten strap to prevent Head Positioner from slipping. (See illustration.)

EASY CARE

Skil-Care's Head Positioner is covered in durable nylon. It may be wiped clean using a damp cloth and spray-type cleanser.