



Headrest

Reorder #	Depth
<input type="checkbox"/> 703115	3"
<input type="checkbox"/> 703116	3¾"
<input type="checkbox"/> 703117	4¾"

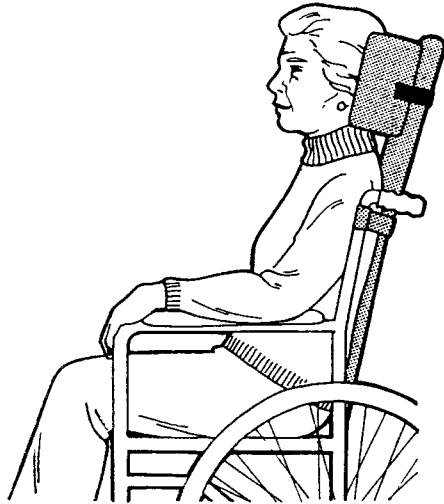

PURPOSE

Laterally stabilizes the resident's head and places it in a position that promotes forward eye contact. This headrest is designed and universally sized to fit all Skil-Care Reclining Wheelchair Backrests as well as extended wheelchair and geri-chair backrests made by other manufacturers.

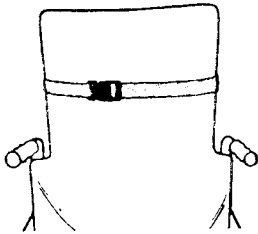

APPLICATION

Position headrest on wheelchair or geri-chair backrest and bring straps behind backrest. Secure buckle and tighten strap to prevent headrest from slipping. (See illustration.)

LAUNDERING

The cover removes for easy laundering. Do not wash or dry the cover at temperature in excess 180°F (82°C). Do not use bleach. Do not launder foam.