


Personal Alarm

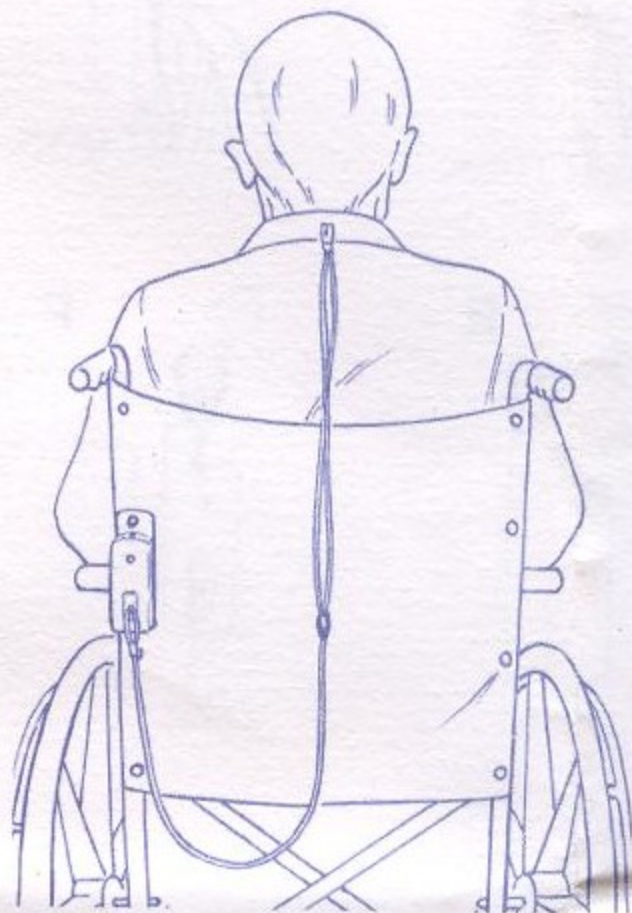

Reorder	Description	Unit
909211	Personal Alarm for Wheelchair and Bed	Each

Purpose

Skil-Care's Personal Alarm alerts caregivers when a resident/patient has left or fallen from the bed or wheelchair.

Cautions

1. The Personal Alarm is not a substitute for proper supervision. Fall-risk and wander-risk residents/patients should be monitored regularly.
2. Check battery each time alarm is used. This is done by setting alarm in high or low volume position and removing magnet. If alarm is weak or fails to sound, replace the 9-volt battery.
3. LED should glow when the alarm is turned on (high or low). If LED fails to glow, replace battery.
4. Periodically check clip to make certain it is attached to resident's/patient's clothing. Patients/residents may be able to defeat the purpose of the alarm by removing the clip.
5. Periodically check to make certain that patient/resident has not removed battery.