

Replacement Sensor Unit for: ChairPro™ Leveling Pad Alarm ChairPro™ Gel-Foam Pad Alarm

Please refer to the model number below when ordering replacement sensor units for Skil-Care's ChairPro Leveling Pad Alarm or Gel-Foam Pad Alarm. Replacement sensor units may be ordered from your medical products supplier.


<u>Model #</u>	<u>Guarantee</u>	<u>Unit</u>
909389	1 Year	Each

Guarantee

Model 909389 is replacement guaranteed against defects in materials and workmanship for a period of one year from date of purchase. Please return defective unit to the dealer from whom it was purchased. The dealer will send the defective unit to Skil-Care and arrange for shipment of a replacement.

Instructions

1. Disconnect "spade" connectors behind chair. (See illustration)
2. Remove cushion or leveling pad from chair and remove sensor unit from pocket on underside.
3. Insert replacement sensor unit. "NOTICE" side must face cushion.
4. Close pocket and place cushion or leveling pad on chair.
5. Place connector cord through space between seat and backrest. Be certain to keep cord away from wheels and other moving parts.
6. Connect "spade" connectors and test alarm.

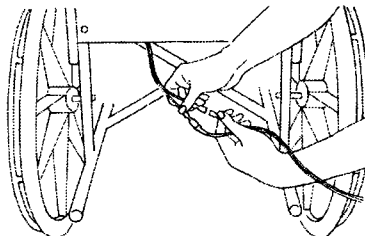

Important

Do not use this sensor unit with any alarm other than a Skil-Care ChairPro Safety Alarm. Skil-Care cannot guarantee that it will operate safely with alarm units other than those provided by Skil-Care Corporation.