


# REPLACEMENT STRAP for RECLINING WHEELCHAIR BACKREST

Reorder #	Size	Unit
<input type="checkbox"/> 703199	Fits all Skil-Care Reclining Backrests	Each

## Purpose

This strap is a replacement for the strap provided as original equipment with Skil-Care's Reclining Wheelchair Backrest. It is adjustable to permit the backrest to provide up to 20 degrees of tilt. It may be used on 16-inch- and 18-inch-wide wheelchairs.

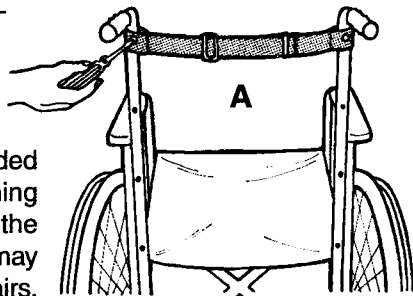

## Warnings

(1) This strap is designed and intended for use only with Skil-Care's Reclining Wheelchair Backrest. Do not use this replacement strap with any product other than a Skil-Care Reclining Wheelchair Backrest. (2) Skil-Care's Reclining Wheelchair Backrest must be used only on wheelchairs equipped with amputee adapters or antitipping devices. Wheelchairs not equipped with such devices may tip over backwards. Amputee adapters and anti-tipping devices are available from your local wheelchair dealer or wheelchair parts supplier.

## Installation Instructions

1. Remove damaged or worn strap.
2. Slip replacement strap over push handles. Adjuster buckle must be facing in direction shown in illustration A. Align holes in strap with the screw holes at the top of the left and right sides of the wheelchair frame.
3. Insert screws through the holes in the strap and screw them into the holes in the wheelchair frame. Tighten screws.

## Adjustment

Use the adjuster buckle to set the backrest at the desired degree of tilt. See illustration B.

