



SLIDE-GUARD CUSHION

Re-Order #	Description	Foundation	Cover Fabric
<input type="checkbox"/> 751210	Gel Cushion	Firm	Polyester
<input type="checkbox"/> 751220	Gel Cushion	Firm	Sheepskin
<input type="checkbox"/> 751230	Gel Cushion	Soft	Polyester
<input type="checkbox"/> 751240	Gel Cushion	Soft	Sheepskin
<input type="checkbox"/> 751260	Foam Cushion	Soft	Polyester
<input type="checkbox"/> 751270	Foam Cushion	Soft	Sheepskin
<input type="checkbox"/> 751280	Foam Cushion	Firm	Polyester
<input type="checkbox"/> 751290	Foam Cushion	Firm	Sheepskin
<input type="checkbox"/> 751265	Foam Cushion	Soft	None
<input type="checkbox"/>			

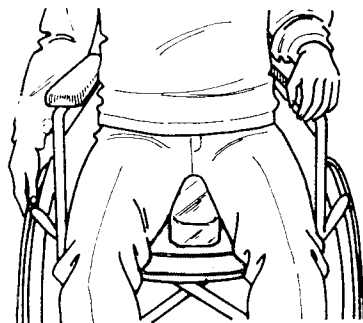

PURPOSE

Skil-Care's SLIDE-GUARD CUSHION helps to position patients who slide down in the wheelchair. In addition, the cushion's pommel acts as a leg abductor. Each Slide-Guard Cushion has ties for the purpose of securing the cushion to the wheelchair. For patient safety, these ties must be fastened to the chair.

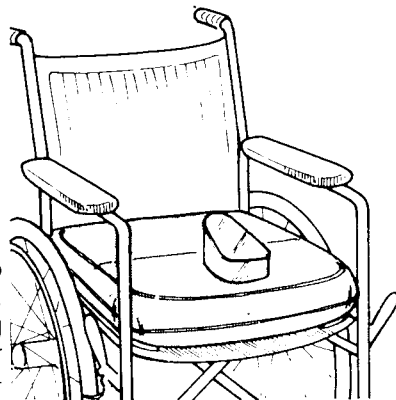

COVER LAUNDERING AND CUSHION CARE

Both the polyester and sheepskin covers remove for laundering. The covers may be washed and dried at temperatures up to 180°F (182°C). Bleaching is not recommended. The vinyl cushion may be cleaned with a mild spray-type cleaner and damp cloth or sponge.