

# SMOKER'S APRON

Re-Order #	Color	Fabric	Unit
<input type="checkbox"/> 906010	White	Skil-Care Firegard <sup>™</sup>	Each
<input type="checkbox"/> 906011	Blue Pattern	Skil-Care Firegard <sup>™</sup>	Each




## Purpose

Skil-Care's Smoker's Apron is designed as an aid in the prevention of accidental ignition of a resident's or patient's clothing. It is not a substitute for proper supervision. Patients or residents in wheelchairs who smoke must be supervised. The Smoker's Apron is intended for wheelchair use only. If the smoker is in a geri-chair, we suggest the larger Geri-Chair Smoker's Apron. The model numbers are 906015 (white) and 906016 (blue pattern).

## Easy to Use

1. Place apron over resident/patient and adjust neck strap so that the top of the apron is above the chest. See illustration above.
2. Wrap the upper two sets of Velcro<sup>®</sup> straps around the left and right metal bars just behind the armrest pads. See illustration A. Secure by pressing hook side against loop side.
3. Wrap lower two sets of Velcro<sup>®</sup> straps around the left and right metal bars just below the front of the wheelchair seat. See illustration B. Secure by pressing hook side against loop side.

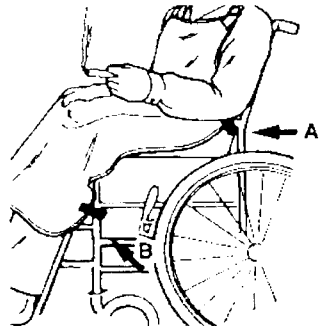

## Easy Care

The Smoker's Apron may be cleaned with a spray-type cleanser and a damp cloth or sponge. Do not launder. This apron is made of Firegard<sup>™</sup> fabric, which can be folded for storage without the risk of developing cracks or pinholes.