


# WHEELCHAIR ARM TRAY LATERAL STABILIZER

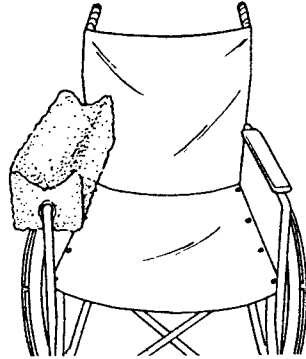

Re-Order #	Description	CoverFabric
<input type="checkbox"/> 706140	Arm Tray Lateral Stabilizer	Polyester
<input type="checkbox"/> 706141	Arm Tray Lateral Stabilizer	Synthetic Sheepskin
<input type="checkbox"/> 706142	Arm Tray Lateral Stabilizer	Spray-and-Wipe Nylon

## PURPOSE

The Wheelchair Arm Tray Lateral Stabilizer, like an armrest bolster, reduces lateral leaning. More than a bolster, it has a trough for improved arm positioning and resident stability. Foam padded for comfort, the Arm Tray Lateral Stabilizer fits all full-arm wheelchairs.

## APPLICATION

1. Open the Velcro closure on cover.
2. Place Arm Rest Lateral Stabilizer on the left or right wheelchair armrest depending on the side to which the resident leans. Left side for left-leaning residents; right side for right-leaning residents.
3. Secure the cover's Velcro closure as snugly as possible.

## COVER LAUNDERING OR CLEANING

Both the polyester and synthetic sheepskin covers may be laundered and dried at temperatures up to 180° F (82°C). Bleaching is not recommended. To reduce the amount of lint that may accumulate in the hook-and-loop closures during laundering, we suggest that you wash and dry the covers with the closures fastened. Do not launder the foam. Nylon covers should not be laundered. Clean them with a spray-and-wipe cleanser and damp cloth.

Replacement covers are available from your Skil-Care dealer. See the Skil-Care price list for replacement cover descriptions and reorder numbers.