


## Wheelchair Lap Trays

Reorder #	Description	Wheelchair Size	Unit
☐705010	Econo -Tray	16" - 18"	Each
☐705019	Nylon Tray	16" - 18"	Each
☐705020	SofTop Tray	16" - 18"	Each
☐705021	SofTop Tray	Pediatric	Each
☐705022	SofTop Tray	20" - 24"	Each
☐705024	SofTop Tray-Bariatric	24" - 26"	Each
☐705030	Dura-Tray	16" - 18"	Each
☐705091	Strap Replacement Kit	All Trays	Set

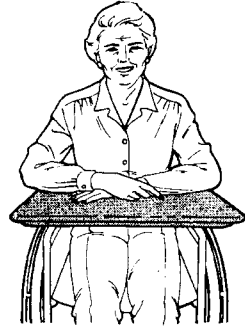

### Purpose

Skil-Care wheelchair lap trays are designed to serve as therapeutic interventions that assist patients in achieving proper body position, balance and alignment. Although most residents can remove Skil-Care trays, there are some who cannot. For them the trays could be considered as restraints. Residents who cannot release standard wheelchair lap trays might benefit from Skil-Care's Lift-Away Tray (models numbers 705015-705025). These are described in the Skil-Care catalog.

### Instructions

1. Place tray on arms of wheelchair arms. Wrap Velcro<sup>™</sup> bands around chair arms and tray. Press the hook material against the loop material to secure.
2. The around-the-back strap is intended to prevent the tray from sliding forward when the resident leans against it. Fasten the strap by bringing it around the back of the wheelchair and passing the end through the buckle. Adjust strap to desired length and secure buckle.

**Note: SofTop Trays are reversible. The unpadded side may be used when a firm surface is required.**

### Easy Care

All trays wipe clean using a damp cloth or sponge. Replacement Velcro<sup>™</sup> bands (model # 705091) are available from your Skil-Care dealer.

**NEW! Skil-Care Firegard<sup>™</sup> Tray Cover (model # 705060) that protects the SofTop Tray against burns caused by matches and cigarettes. See Skil-Care catalog for details.**